

# A-Tagger

## Instructions for use




**1** Flip the applicator pin out.




**2** Place the A-Tag halfway onto the applicator pin.

**3** Flex the stem of the A-Tag to a horizontal position.

**IMPORTANT:**  
The A-Tag has a sharp point and care should be taken to avoid an injury.

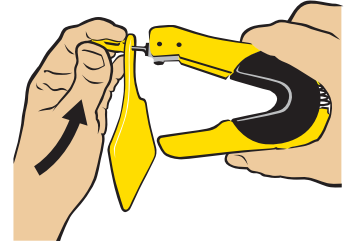


**4** Slide the A-Tag completely onto the Applicator Pin.

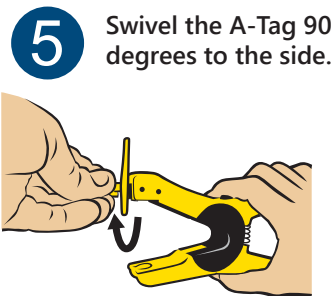


**5** Swivel the A-Tag 90 degrees to the side.

**6** Flip the Applicator Pin in. The tag is now in the application position.

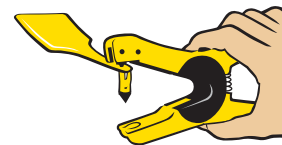

**7**

Position the sharp tag point between the two main cartilage ribs in ear. Do not use existing tag holes.

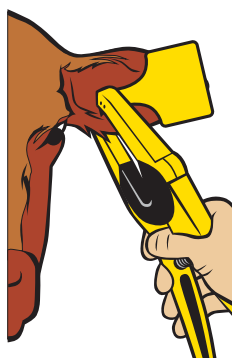

**8** To apply; squeeze the handles together. Pull the Applicator down with a firm, positive action away from the animal. Do not release the handles.

